# YHP Laser Safety Glasses

## USER MANUAL

<b>REF</b>	G1462
<b>MD</b>	<p><i>Class I</i>  <i>In accordance with rules 1 set out Annex VIII</i>  <i>Marked according to Annexes II and III</i>  <i>MDR (EU) 2017/745</i></p>

	<p>LaserPair Co., LIMITED          NO. 1, PINGJI ROAD, NANWAN STREET, BUJI          TOWN LONGGANG, SHENZHEN518112, P.R.CHINA          Url. <a href="http://www.laserpair.net">www.laserpair.net</a> Mail. <a href="mailto:sales@laserpair.com">sales@laserpair.com</a>          Tel. +86 755 8452 9966 +86 755 2840 2669</p>	<b>EC REP</b>	<p>DOMINO SRL          VIA VITTORIO VENETO 52,          31013, CODOGNE' (TV), ITALY          Url. <a href="http://www.globuscorporation.com">www.globuscorporation.com</a>          Mail. <a href="mailto:info@globusitali.com">info@globusitali.com</a>          Tel. +39 04387933</p>
---	--	---------------	---

YHP laser safety glasses is designed for absorbing special laser wavelength to provide safe working condition to our customers. The lens of the glasses are made of macromolecule and laser absorber. The optical properties of the lens meet all demand of the standard DIN EN 207.

### Technical Characteristics:

Type	Lens Color	Lasers	Protect Wavelength (nm)	Protection Level	Image
LP-YHP	GREEN	DIODES YAG	808nm 905nm 980nm 1064nm	800-1100nm OD5+ 1060-1070nm OD7+	

Marking: 800-1100nm DIR LB5 1000-1070nm IR LB7 LP S CE

### Marking explanation:

800-1100nm DIR LB5 LP S CE

LP Manufacturer code, D Continuous wave laser, I R Pulsed laser

800-1100nm - Protect wavelength


LB5 - Scale number

S - Mechanical resistance symbol

Warning:

- 1) The laser safety glasses can not be used against other kind of lasers rather than the above mentioned wavelength
- 2) Do not look into the laser beam directly
- 3) Please clean and disinfect the glasses with soapy water or Johnson Glass Cleaner etc.,
- 4) Do not store the glasses under sunlight, high temperature directly
- 5) Please note these eye-protectors are only intended to give protection against accidental radiation
- 6) Please note if these eye-protectors and filters against laser radiation which have been damaged, have scratched oculars or which have undergone a color change shall not be used any more
- 7) Please note there is also a risk of exposure to laser radiation due to reflection from reflective parts (including eye-protectors), tilting or misalignment of optical components.
- 8) Please note in the areas where there is a risk of exposure to laser radiation, an appropriate eye-protection should be worn by all persons
- 9) Please check the transmission curves of these glasses in appendix file
- 10) Accessories: Cleaning fabric, Giftbox
- 11) Please note that optical class 3 oculars are not intended for long term use
- 12) Please note that any materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals
- 13) Please note that scratched or damaged oculars or damaged oculars should be replaced
- 14) Please be aware that the recognition of the warning lights or warning signals can be impaired
- 15) We recommend a use for a period of 3 years. The duration of use depends on various factors such as use, cleaning, storage and maintenance. Frequent inspections and replacement if it is damaged are recommended
- 16) Customer further risk analysis on their product: Frame break down, Filter break down, Filter color changed, use for wrong lasers, etc.
- 17) OD curve of the lens

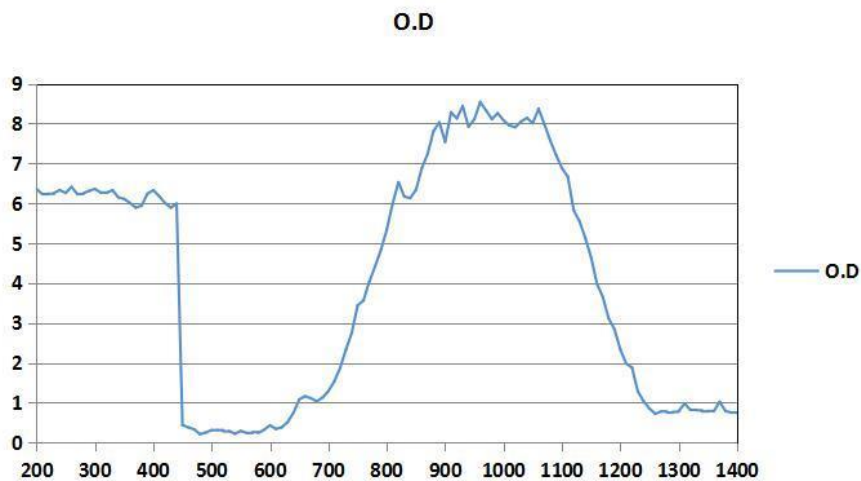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